




Proficiency Test	Assigned value (mg/L)	Result (mg/L)	Z-score
BIPEA Allergens in Industrial Process Water 77D - Industrial Process Water – 21 - 1077 – June 2026	0.3	0.3	0.00
BIPEA Allergens in Industrial Process Water 77D - Industrial Process Water – 21 - 1177 – June 2026	NOT DETECTED	<0.1	-
BIPEA Allergens in Industrial Process Water 77D - Industrial Process Water – 21 - 1277 – June 2026	0.5	0.5	0.00
BIPEA Food Allergens - Milk 77C - Infant Formula – 23 - 0777 – April 2026	0.4	0.3	-0.5
BIPEA Food Allergens - Milk 77C - Infant Formula – 23 - 0877 – April 2026	NOT DETECTED	<0.1	-
BIPEA Food Allergens - Milk 77C - Infant Formula – 22 - 0977 – April 2026	0.9	0.7	-0.67

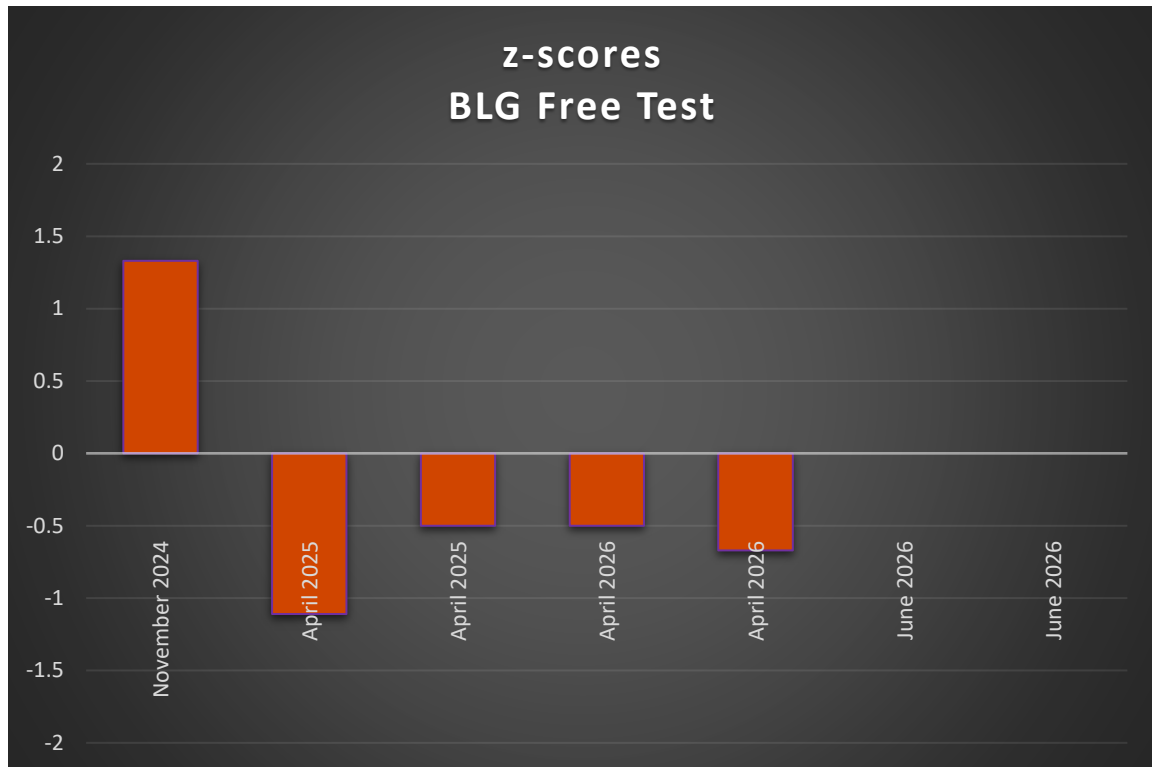

BIPEA Food Allergens - Milk 77C - 20 - Infant Formula - 21 - 0777 - April 2025	NOT DETECTED	<0.1	-
BIPEA Food Allergens - Milk 77C - 20 - Infant Formula - 21 - 0877 - April 2025	1.2	0.8	-1.11
BIPEA Food Allergens - Milk 77C - 20 - Infant Formula - 20 - 0977 - April 2025	0.5	0.4	-0.50
BIPEA Food Allergens - Milk 77C - 19 - Infant Formula - 20 - 0777 - November 2024	NOT DETECTED	<0.6	-
BIPEA Food Allergens - Milk 77C - 19 - Infant Formula - 20 - 0877 - November 2024	NO VALUE WAS ASSIGNED	<0.6	-
BIPEA Food Allergens - Milk 77C - 19 - Infant Formula - 19 - 0977 - November 2024	0.4	0.6	1.33

Proficiency Test	Assigned value (mg/kg) according to Allergen-Shield BLG A1348/A1396	Range for $ z \leq 2$ according to Allergen-Shield BLG A1348/A1396	Result (mg/kg)	Recovery (%)
FAPAS Milk in Infant Breakfast Cereal 27475A June - August 2026	3.74	1.87 - 5.60	2.85	76.2
FAPAS Milk in Infant Breakfast Cereal 27475B June - August 2026	NOT DETECTED	-	<0.1	-
FAPAS Milk & Gluten in Infant Soya Formula 27463A March - May 2026	NOT DETECTED	-	<0.1	-



FAPAS Milk & Gluten in Infant Soya Formula 27463B March – May 2026	2.72	1.36 – 4.07	DETECTED	-
FAPAS Gluten, Milk & Egg in Cooked Biscuit 27459 January – March 2026	2.06	1.03 – 3.09	2.41	117.0

Proficiency Test	Assigned value (mg/kg) according to R-Biopharm Ridascreen Fast β -Lactoglobulin R4912	Range for $ z \leq 2$ according to R-Biopharm Ridascreen Fast β -Lactoglobulin R4912	Result (mg/kg)	Recovery (%)
FAPAS Gluten, Milk & Egg in Cake Mix 27438A July – August 2025	2.12	1.06 – 3.18	2.06	97.2
FAPAS Gluten, Milk & Egg in Cake Mix 27438B July – August 2025	3.44	1.72 – 5.16	4.00	116.3
FAPAS Milk & Gluten in Plant Based Milk Alternative 27445A September – October 2025	NOT DETECTED	-	<0.1	-
FAPAS Milk & Gluten in Plant Based Milk Alternative 27445B September – October 2025	3.20	1.60 – 4.80	3.30	103.1
FAPAS Milk in Infant Breakfast Cereal 27435A June – August 2025	1.72	0.858 – 2.57	2.26	131.4
FAPAS Milk in Infant Breakfast Cereal 27435B June – August 2025	NOT DETECTED	-	<0.1	-



Proficiency Test	Assignment	Result (mg/kg)	Satisfactory results / results agreeing with assignment (%)
FAPAS Milk & Gluten in Infant Soya Formula 27422A February – March 2025	DETECTED	DETECTED	-
FAPAS Milk & Gluten in Infant Soya Formula 27422B February – March 2025	NOT DETECTED	<1.50	-
FAPAS Gluten, Milk & Egg in Cooked Biscuit 27418 January – March 2025	DETECTED	DETECTED	-
FAPAS Milk in Infant Breakfast Cereal - 27398A - June – August 2024	NOT DETECTED	<2.0	-
FAPAS Milk in Infant Breakfast Cereal – 27398B - June – August 2024	DETECTED	DETECTED	-